

VERCISE™
Deep Brain Stimulation Systems

USING YOUR VERCISE™ PC AND VERCISE GEVIA™ REMOTE CONTROL

A QUICK START GUIDE FOR PATIENTS AND CAREGIVERS



Rx ONLY

CAUTION: Federal law restricts this device to sale, distribution and use by or on the order of a physician.

92354407-01
Content: 92354409 REV A

An Overview of the Remote Control



How Can I Start or Stop Stimulation?



To get started, press the “unlock” button on the right-hand side of your remote.



Now you can turn stimulation on by pressing the red button in the center of your remote.

To turn stimulation off, simply press the red button again.

How Can I Increase or Decrease Stimulation?

If your doctor allows you to change your stimulation level, this is easily done on your remote:



First click the right-hand arrow to see the main menu.

Then select "stimulation".



You can then use the "plus" or "minus" buttons beside the on/off button to adjust your stimulation to the right level for you.

How Can I Access or Change Stimulation Programs?

If your doctor has set up different stimulation programs for you, you can access these in just a few clicks:



EITHER: press the “Programs” button on the left side of your remote.



OR: Click the right-hand arrow to see the main menu.

Click the right-hand arrow a second time to enter the "Programs" menu.

Click the right-hand arrow a third time to enter the "Change Program" menu.



From there you can **navigate** through the different programs with the up and down arrows.




Select your desired program by clicking on the right hand arrow one last time.

How Can I Charge My Remote?



To charge your remote, simply connect it to the **USB power supply** and plug into a **plug socket**.

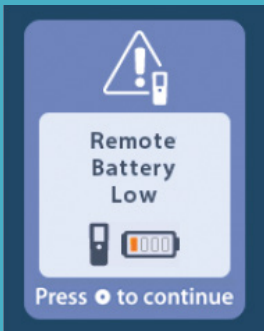

Whenever the remote is **low on battery**, it will show you a reminder to recharge.

GUARANTEES

Boston Scientific Corporation reserves the right to modify, without prior notice, information relating to its products in order to improve their reliability or operating capacity.

TRADEMARKS

All trademarks are the property of their respective holders.

ADDITIONAL INFORMATION

For indications, contraindications, warnings, precautions, adverse events summary, sterilization, component disposal and storage and handling, refer to the *Information for Patients DFU* or *Information for Prescribers DFU* for your Boston Scientific DBS System. Physicians: For other device specific information not included in this manual or labeling symbols, refer to the appropriate DFU for your Boston Scientific DBS System as listed on your *DBS Reference Guide*.

Boston Scientific

Advancing science for life™



Legal Manufacturer

Boston Scientific
Neuromodulation Corporation
25155 Rye Canyon Loop
Valencia, CA 91355 USA
+1 (866) 789-5899 US & Canada
+1 (866) 789-6364 TTY
+1 (661) 949-4000 International
www.bostonscientific.com
Email: neuro.info@bsci.com



Australian Sponsor Address

Boston Scientific (Australia)
Pty Ltd
PO Box 332
BOTANY
NSW 1455
Australia
Free Phone 1800 676 133
Free Fax 1800 836 666



EU Authorized Representative

Boston Scientific Limited
Ballybrit Business Park
Galway, Ireland
T: +33 (0) 1 39 30 97 00
F: +33 (0) 1 39 30 97 99